HZ-3705

Fom

Electric Beauty Bed / Camilla eléctrica de estética / Table d'esthétique électrique /
Lettino Elettrico Beauty / Elektrische Kosmetikliege

EN	2
ES	4
FR	6
IT	8
DE	10

DDUUEETT EUROPE would like to thank you for choosing Sonia products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.

- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.
- Never expose your hands or any part of your body to any hole or vacuum on the bed while it is in operation.


CONTENTS

- Electric Beauty Bed x 1 unit
- Headrest x 1 unit
- Armrests x 2 units
- Instruction manual x 1 unit

TECHNICAL SPECIFICATIONS

Dimensions	186x78x63-85 cm
Net weight	75 kg
Max. Load	150 kg
Motors	1
Upholstery	PU
Voltage	220-240 V
Input frequency	50/60 Hz

SKETCH AND DIMENSIONS

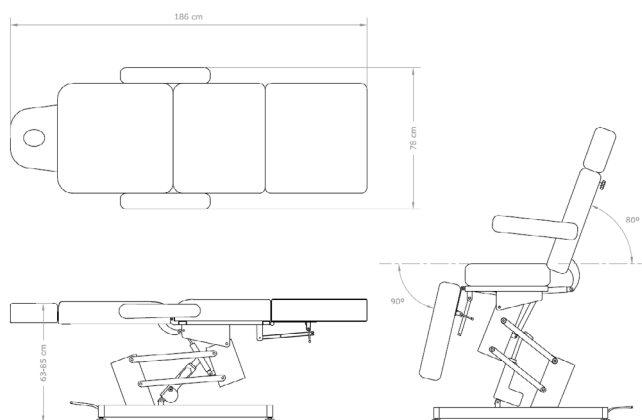
INSTRUCTIONS FOR ASSEMBLY

- Unscrew the clamping knob and insert the headrest into the backrest of the main body of the bed. Adjust the headrest to the desired height and turn the knob clockwise.


- Insert the armrest in the correct position so that the pin fits with the plastic part of the stretcher body and push it in until the stop.
- Turn the armrest until it is in a more or less vertical position and you are not allowed to continue turning.
- Pull out until the stop again and then continue turning until the armrest rests in a horizontal position.
- Perform the same procedure on the 2 armrests.

OPERATION

- Connect the bed to an electrical outlet using the power cord.
- Hand controller
 - Adjusting height
 - Use the buttons  and  of the control knob to adjust the stretcher height.
- Manual functions
 - Adjusting backrest tilt

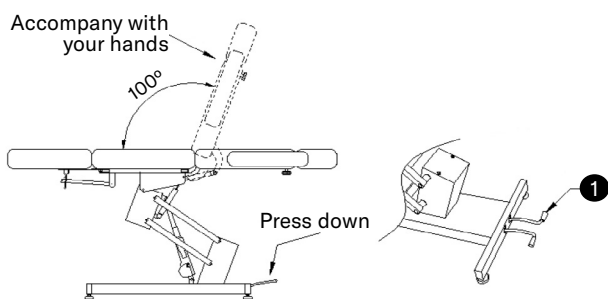
- Press lever **1** downwards with your foot, while tilting the backrest with your hands.
- When you release lever **1**, the backrest will stay in its current position.
- Press lever **1** downwards again with your foot and the backrest will return to its 100 position relative to the horizontal line.
- Adjusting seat tilt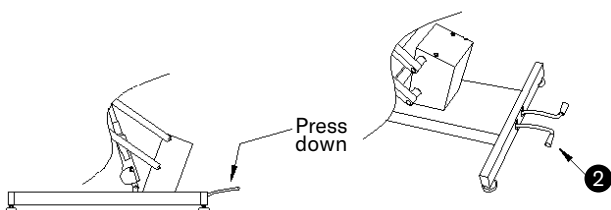

- Press lever **2** downwards with your foot, while tilting the backrest with your hands to lean your seat backwards.
- When you release lever **2**, the seat will stay in its current position.
- Press lever **2** downwards again with your foot and the seat will return to its horizontal position.

- Adjusting footrest tilt

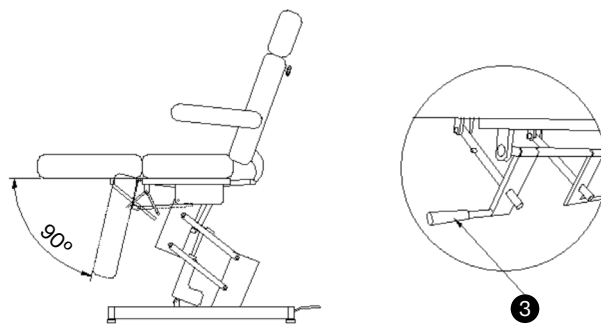- Use lever **3** located under the footrest. Push it towards the centre of the stretcher and the footrest will tilt downwards.
- When you release lever **3**, the footrest will stay in its current position.
- To tilt the footrest up, push it up to the desired position.

CLEANING AND MAINTENANCE

- Before cleaning the device, unplug it from the mains.
- Clean the device with a soft, dry cloth. Do not use liquids to clean it.
- For maintenance or repairs of any part of the product, please contact your distributor or specialised technician.

RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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