



HZ-3873A User Manual

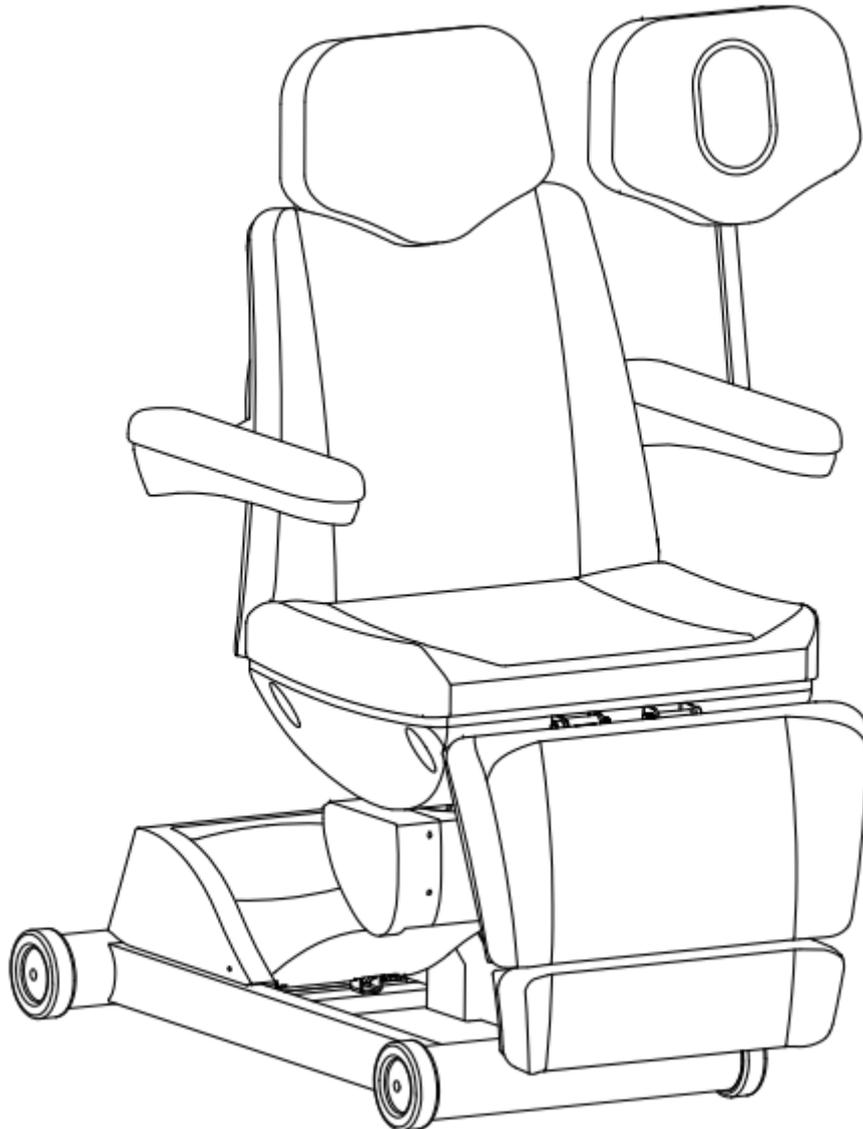

● Technical data:

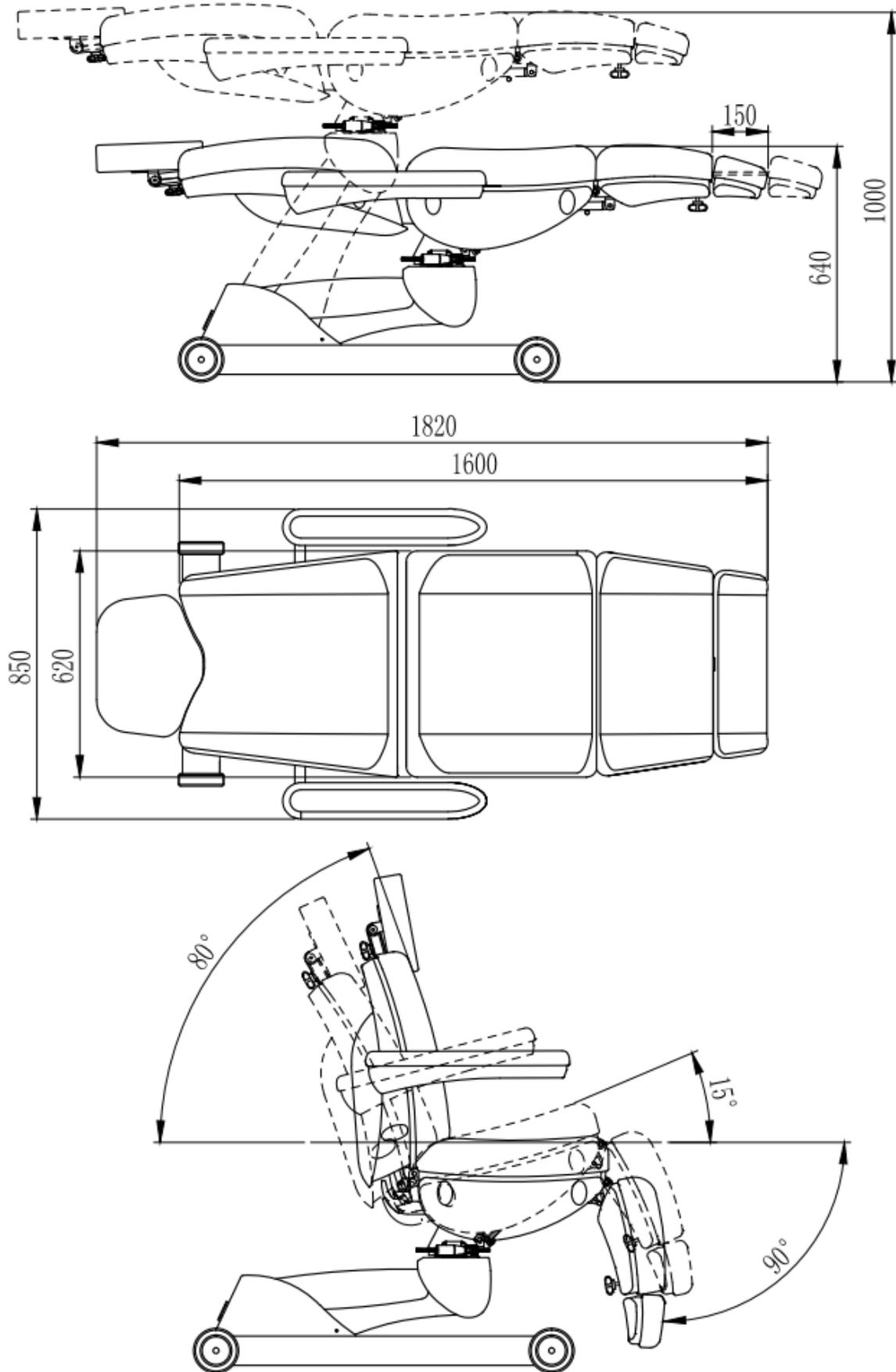


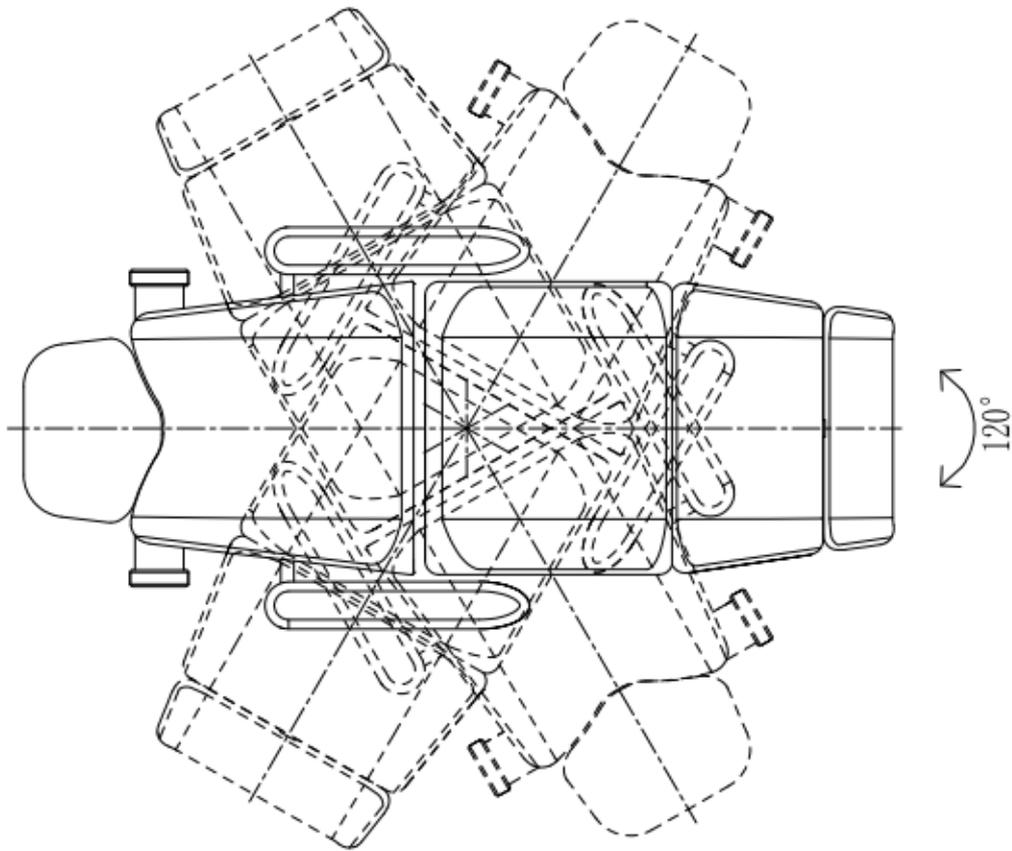

Technical data:


| | |
|---------------------|----------------|
| Size | 1820*850*640mm |
| Load | 200kg |
| Net weight | 85kg |
| Adjust height range | 620~980mm |
| Padding | sponge |
| Leather | PU/PVC |

Accessories list:
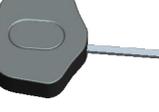
| Name | Picture | PCS |
|----------------------|---|-----|
| left armrest |  | 1 |
| right armrest |  | 1 |
| headrest |  | 1 |
| headrest (with hole) |  | 1 |


Assembly instructions

- Install the armrests, connect the bar of left and right armrests with the inner pin of the bed, rotate the armrests by 90 degrees counterclockwise.
- Install the pillow, connect it to the bed, loose the knob to adjust the height of the pillow.

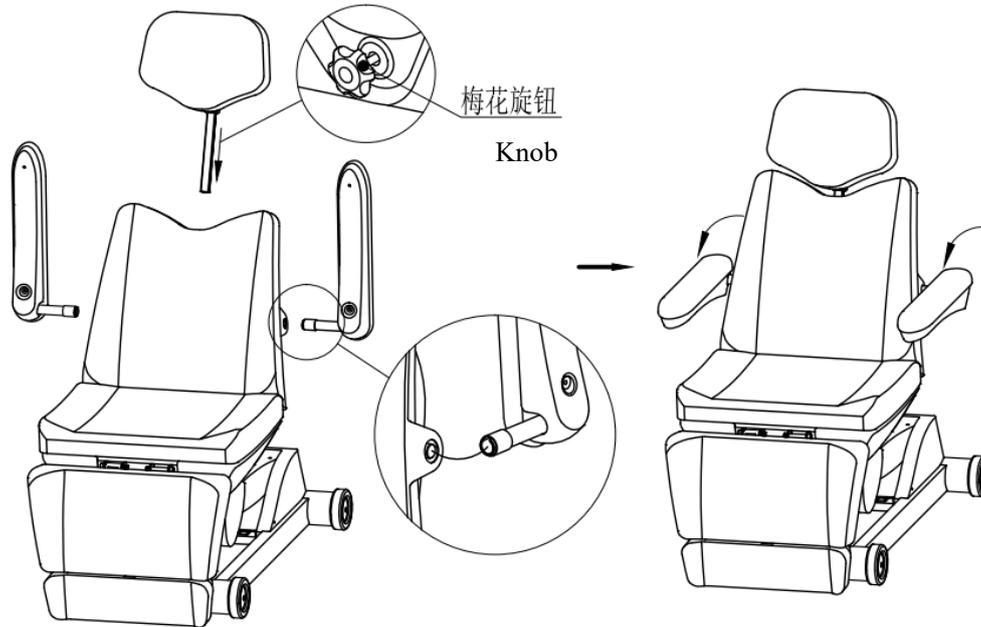


● Use Instructions

- Foot controller use instructions.

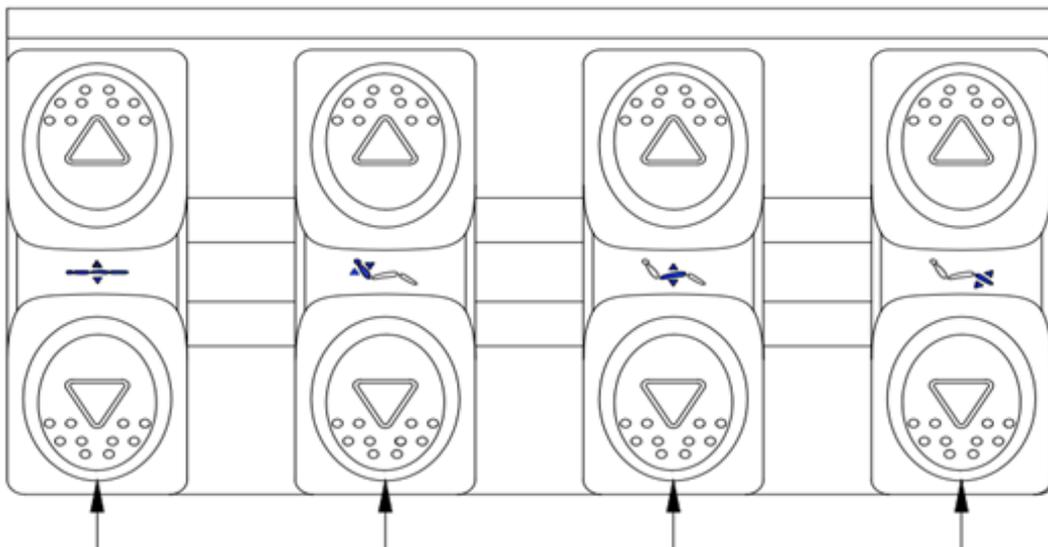

- **Intrc** adjust the whole bed up and down.
- an** adjust the backrest up and down.
- pa** adjust the incline angle for the seat.
- adjust the footrest up and down

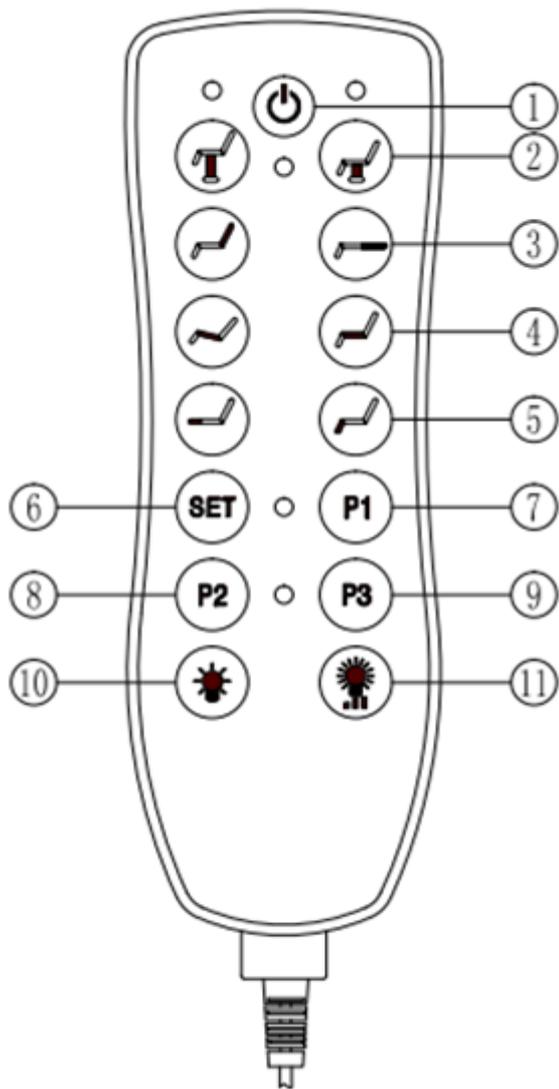

Press button ① the bed will power on, you can use all functions; power off, the bed will come back to its original state, press again, it will stop to come back.

Press button ② to adjust the whole bed up and down.

Press button ③ to adjust the backrest up and down.

Press button ④ to adjust the incline angle for the seat.

Press button ⑤ to adjust the incline angle for the footrest .

Press button ⑥-⑨ come to the auto keys, press any of the keys to be adjusted to the pre-setting state.

Press button ⑩ to turn on the color light and press once to change the light color.

Press button ⑪ to adjust the luminance of the color light.

Remarks:Automatic mode setting,to set up the state for the height ,the lenght for the seat、 backrest and footrest, to long press button SET until the blue light turn on to start the memory funtion, to press button P1 to set the first state ,the same method for P2 and P3 to set the second and third state.


● The height adjustment

Method One: Turn on the power switch and the manual power switch on the hand controller, to

press the button  , to rise up the whole bed , to press the button  , to fall down the whole bed.

Method Two: Turn on the power, then step on the pedal position  , to rise up or fall

down the whole bed.

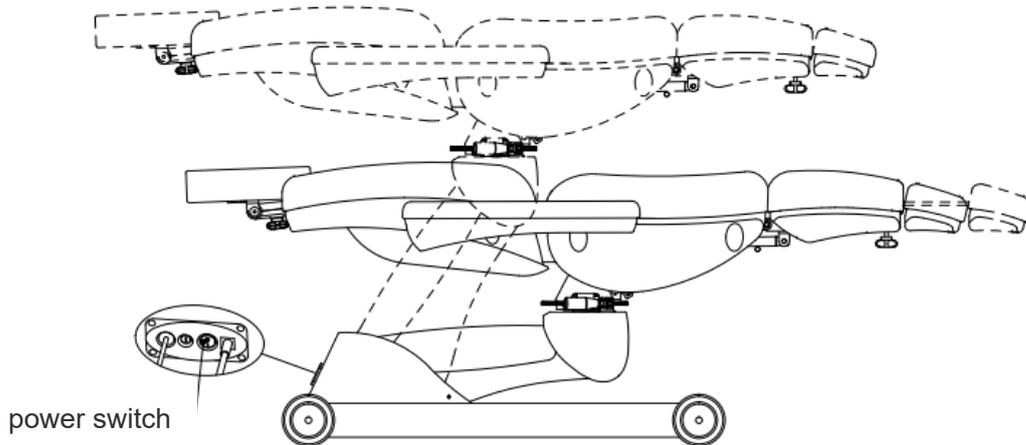


● The Backrest adjustment

Method One: Press button  on the hand controller to adjust the backrest up , Press button

 on the hand controller to adjust the backrest down.

Method Two: Step on the pedal position  to adjust the angle for the backrest.

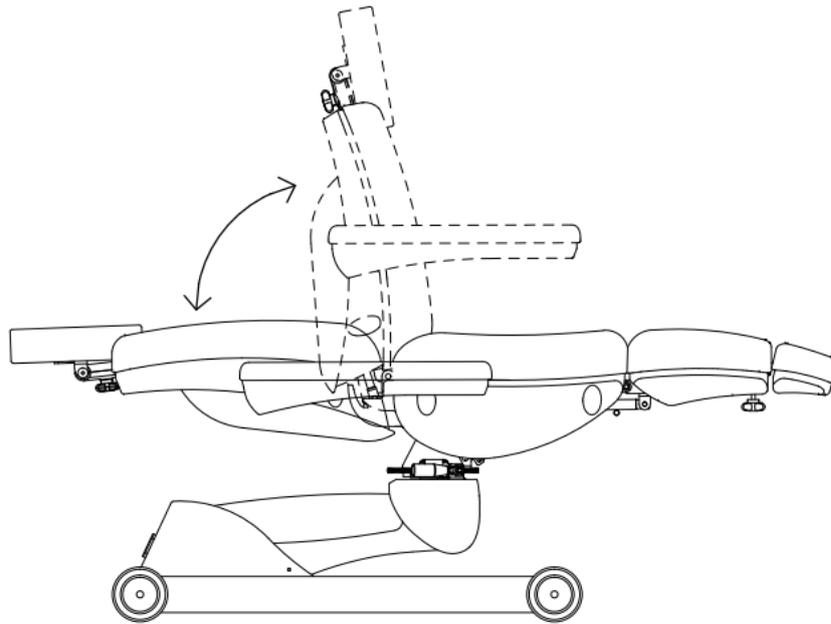


● The seat adjustment

Method One: Press button  on the hand controller, to increase the angle of the seat,

Press button  on the hand controller, to decrease the angle of the seat.

Method Two: Step on the pedal position , to adjust the angle for the seat.

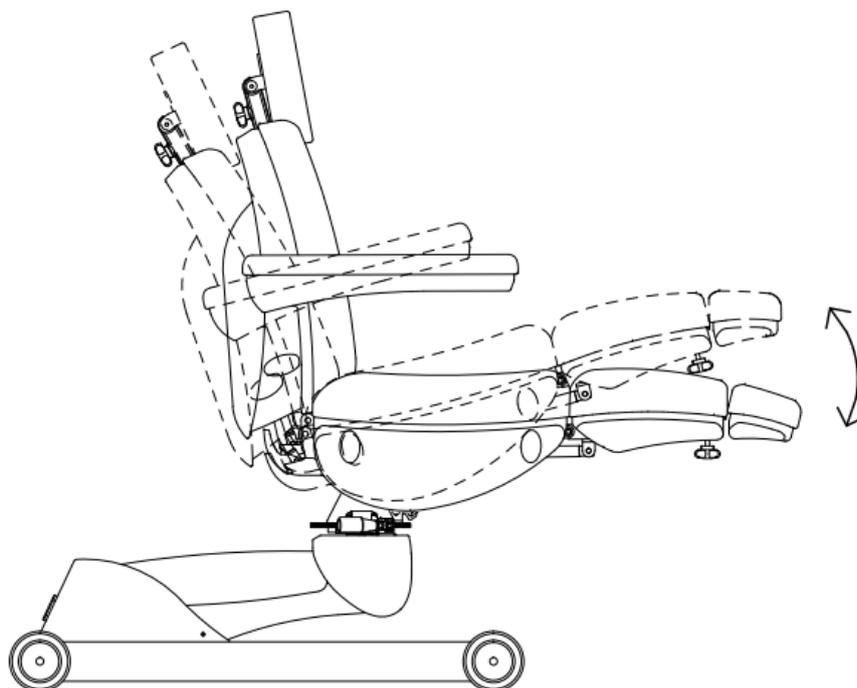


● The footrest adjustment

Method One: Press button  on the hand controller, to increase the angle of the footrest,


Press button  on the hand controller, to decrease the angle of the footrest.

Method Two: Step on the pedal position , to adjust the footrest up and down.

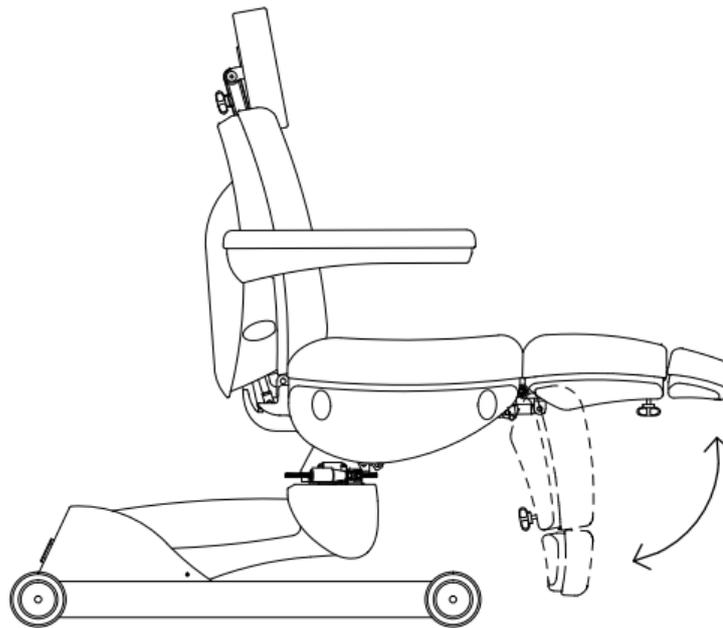


● Footrest extension adjustment

To adjust the extension of the footrest by rotating the knob under the footrest, as below picture shown.

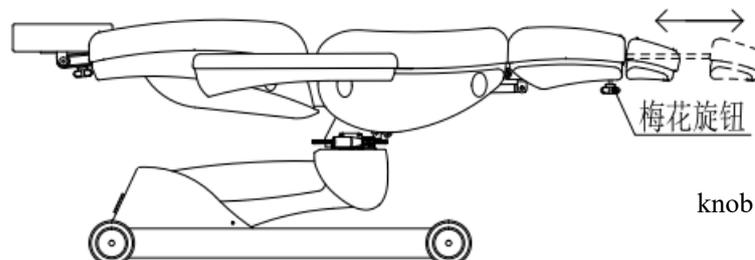

● Rotate adjust

Pull the handle to separate the rotating adjusting block from the rotating seat, to push the bed to rotate clockwise or counterclockwise $0^{\circ} \sim 120^{\circ}$. After adjusting the rotation Angle of the bed, pull the handle to make the rotating adjusting block get stuck in the rotating seat to fix the bed.

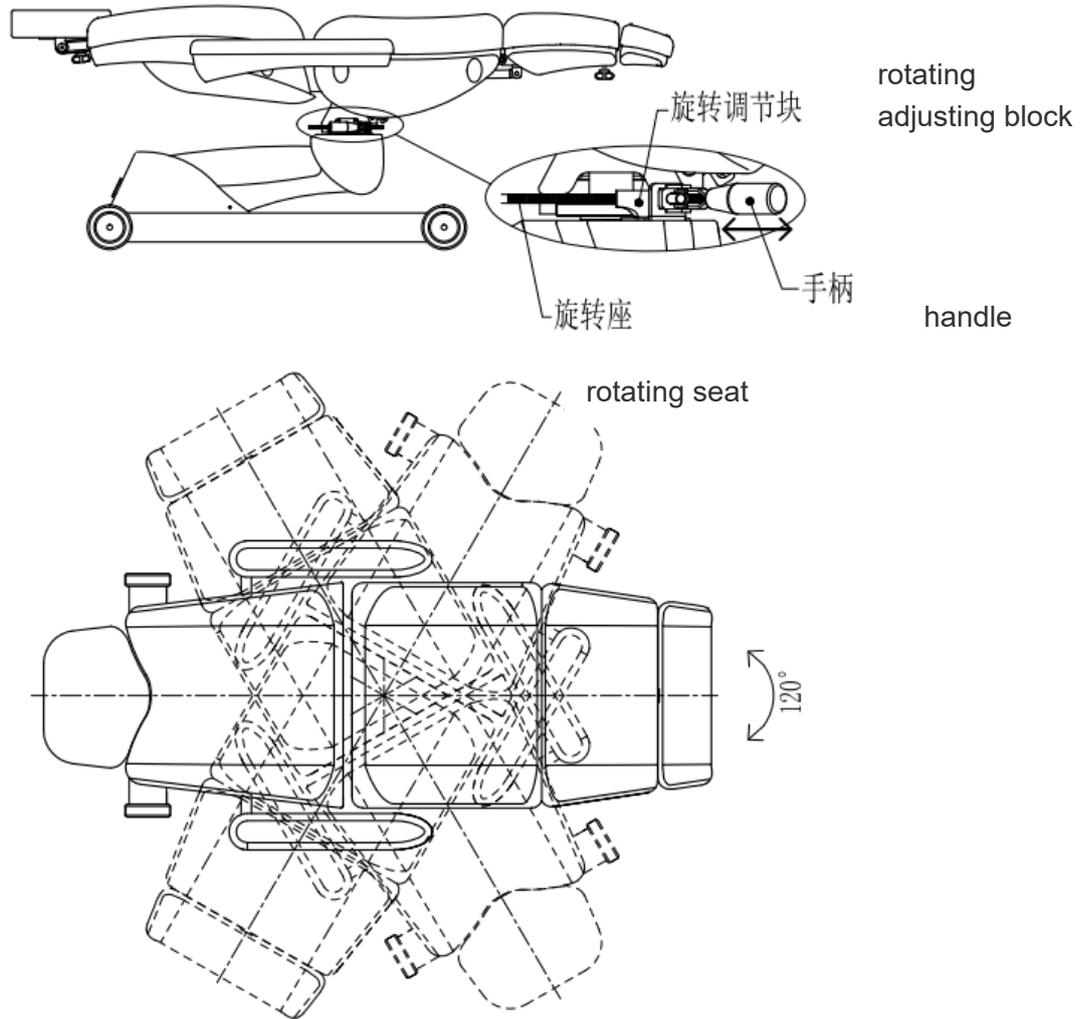

● The pillow adjustment

The height of the pillow can be adjusted by loosen the knob A. You can rotate or flip over the pillow by loosen the knob B, as shown in below picture.




The suggestive instructions

1. This electric bed uses a three-wire power cable. Meanwhile it should be inserted into the three-hole outlet where there is a ground electrode. And it should be assured that it can be

connected to the ground electrode smoothly so that it can prevent people from getting electric shock while functioning normally.

2. All the plugs, buttons and other components connected to the power cable must not dip in the water or other liquid.
3. It is forbidden to put your hands, feet, head or other part of the body into the the gap show as below ①②③④ while the bed is running .
4. It is prohibited to use when plugs, buttons and other components connected to the power cable are broken.
5. It should be noticed that the frequency is 6min/h when the motor of this bed is used.
6. The maintenances should not be dealt with without official training.

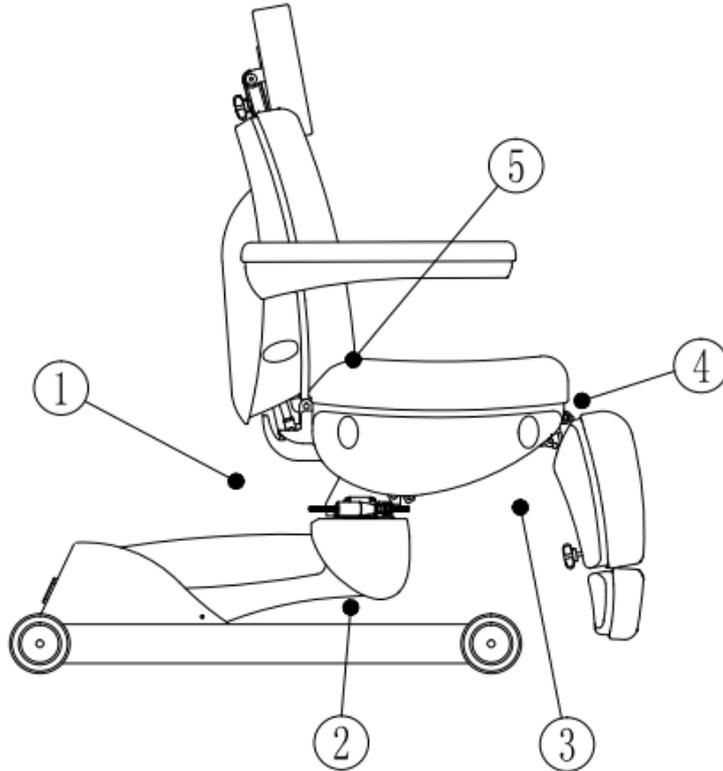

Attentions

1. Someone with osteoporosis or heart problem should not use this massage chair.
2. Women being pregnant or in menstruation should not use this massage chair.
3. Someone inured or with disabilities should not use this massage chair.
4. Children under 14 years old without any supervision and someone unconscious should not use this massage chair.
5. Breakages of the power cable and plug should be eliminated before it's normally used. Breakages should be dealt with by someone professionals or service providers.
6. The voltage should accord with the safety voltage before it's officially used. And the wrong voltage may lead to the breakdown of the product.
7. Wet hand is not allowed to touch power to avoid shock.
8. The product should not plug in before it's already installed.
9. The power cable should be put away so that pedestrian will not trip up and fall.
10. The appearance of the product should cleaned by the use of the soft towel with diluted neutral detergent.
11. It should be unplug when it's not used.

12. Do not dismount this product without any authorization. Please inform the official service providers to deal with the maintenances

Maintenance

1. Daily cleaning
You can wipe out the dust by using soft dry towel.
2. Eliminating the stubborn dirt
As we all know, it is hard to eliminate the stubborn dirt by using normal dry towel. In this case, you can use soft towel with diluted neutral detergent to clean it first. And then you can use dry towel to scrub.
3. It's prohibited to use benzene, diluent, gasoline or other solvent.
These solvents may damage the leather or it may cause the corrosion of the products' appearance.
4. Storage and operating environment
Don't put the product under the strong sunlight and moisture environment. Otherwise it may cause the damages of the product.
5. It is allowed to use this product under the temperature from -5°C to 50°C and the humidity from 45% to 85%.

Fault diagnosis

If there is something wrong when you use the product, please try to check and eliminate the wrong operation as followings.

| Phenomenon | Diagnosis | Operation of elimination |
|--|---|--|
| Press the pedal but without any responses. | <ol style="list-style-type: none"> 1. Controller power switch start or not. 2. To check if the pedal has reached to maximum. 3. To check if the power plug is connected. 4. To check if each connecting plug of the transformer is connected. | <ol style="list-style-type: none"> 1. Try to re-start the controller power switch. 2. Try other buttons workable or not. 3. Disconnect the power plug and re-plug it again. 4. Re-plug and connect it again. |
| Letting out noises and shaking while going up and down | Screw loosen | Tighten all the screws |
| The light is still off although the switch is on. | The fuse has burned out. | Replace the fuse. |

◆ **Remarks: It will be not informed if the improvements of the product do not influence appearance and function**