VISMARA  
BEAUTY & SPA DESIGN

# Podologica Saint Barth

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## Caratteristiche idrauliche / *Hydraulic data*

---

COLLEGAMENTI ACQUA / *WATER SYSTEM CONNECTION*: 1/2"

COLLEGAMENTO SCARICO / *DRAIN CONNECTION*: 40 mm

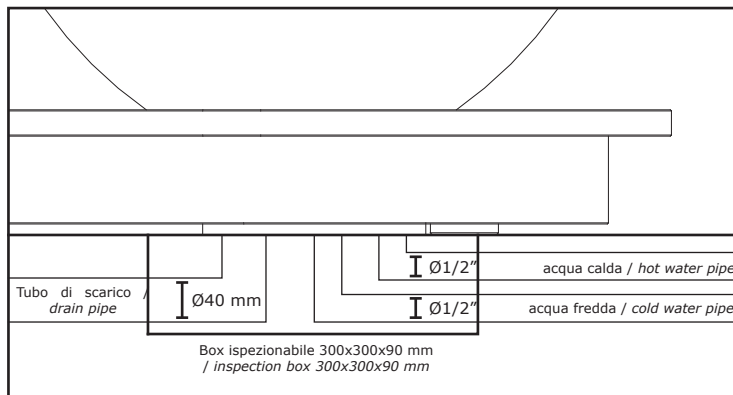
CONSUMI DI ACQUA / *WATER CONSUMPTION*: From 11 to 20 L/min

PRESSIONE D'ESERCIZIO / *PRESSURE IN OPERATION*: 1.5/4.5

## Collegamento idraulico / *Water system connection*

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*The connection of the Podiatric chair Saint Barth has to be executed by qualified professionals, who should issue a certification of their work, and has to follow the scheme below.*

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Rigid or semi-rigid tube of 40 mm diameter. It is advisable to guarantee a gradient of 1,2 cm per meter in order to ensure a proper drainage.

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*The best solution for the water system connection is to create a supply and drainage system directly in the floor.*

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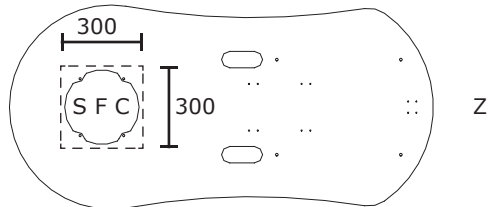
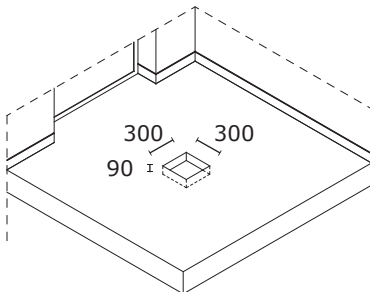
*The installation of Saint Barth podiatric chair has to be executed by qualified personnel. Before proceeding with the connection to the power system please verify that it is protected by an RCD (0,03) and that the grounding system complies with the specific country's regulations on the matter.*

*The Podiatric chair Saint Barth has to be fixedly connected to the power supply (with a H05 - 3x2,5 mm tripolar cable with a section of 2,5 mmq) in compliance with the product voltage and far from electrical zones 0,1,2,3 and from any water spurts.*

*Warning: the non-compliance, even partial, of the present safety indications could entail the functioning of the product. If any breach of these indications were to occur both the manufacturer's responsibility and the product's warranty would be subject to forfeiture for any direct or indirect damage caused by the product malfunction.*

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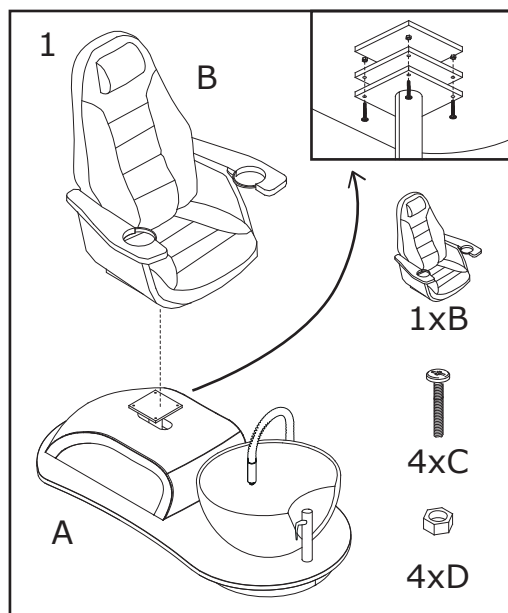
*Please check that the packaging is intact and that the Podiatric chair Saint Barth has not been damaged during its delivery.*

*The installation has to be carried out according to the present regulations regarding the matter, therefore it is advisable to have qualified staff to install it and please do comply to the following indications.*

*In case of failure in compliance to these indications the warranty is subject to forfeiture.*

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
*Place the base (A), then lower the seat (B) on the dedicated plate, making sure that holes placed on both the plate and the seat match.*

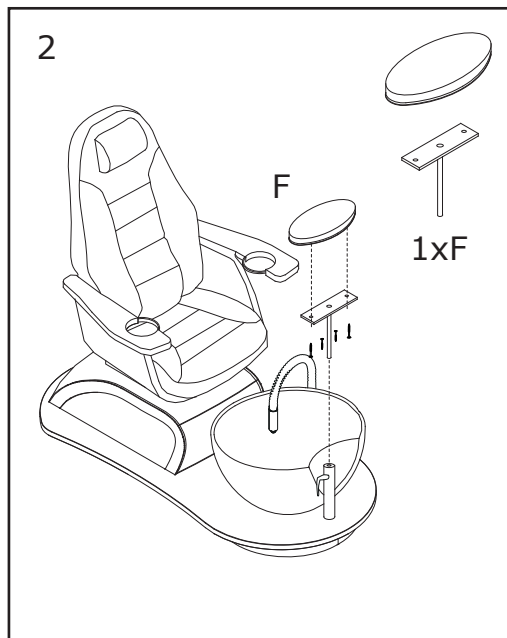
*Then fix the two elements with 4 M8 screws (C) and the respective bolts (D).*

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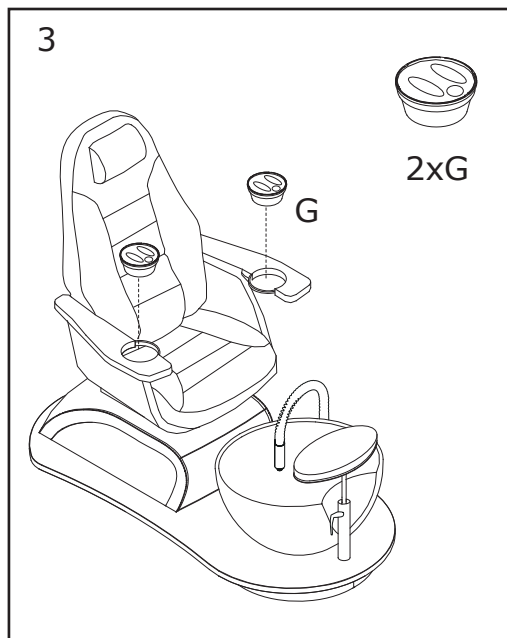*Screw the metal support to the footrest (F).  
Then slide the footrest to the mounting bracket.*

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*Place the manicure bowls (G) in the dedicated slots located in the seat arms.*

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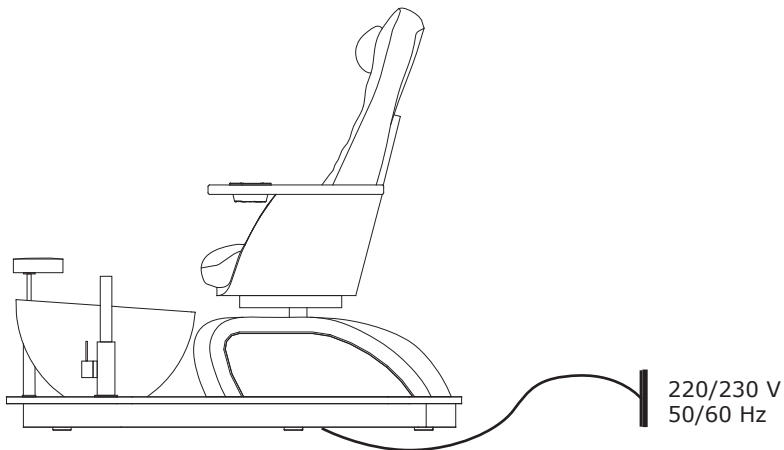
*The Podiatric chair Saint Barth is equipped with the dedicated cable for the connection to the power system.*

*Please verify that the voltage of the power supply system is 220/230V 50/60Hz.*

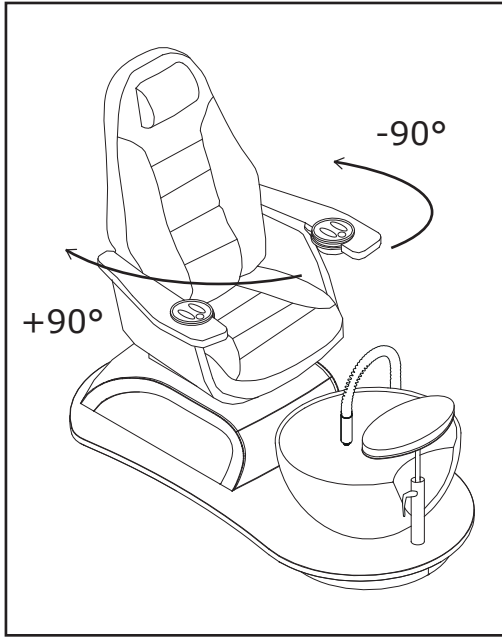
*It is necessary to also install a safety switch in the system connected to the podiatric chair.*

*The maximum energy consumption of the Podiatric chair in operation is 500 Watt.*

*If it is necessary to modify the connection cable in order to connect the Podiatric chair directly to the power system please refer to qualified personnel.*

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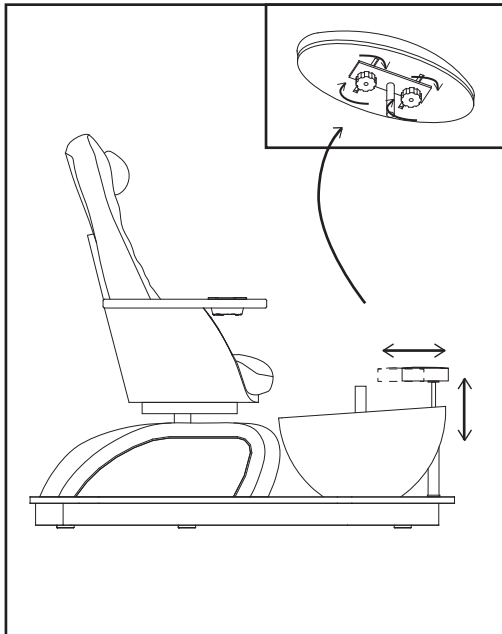
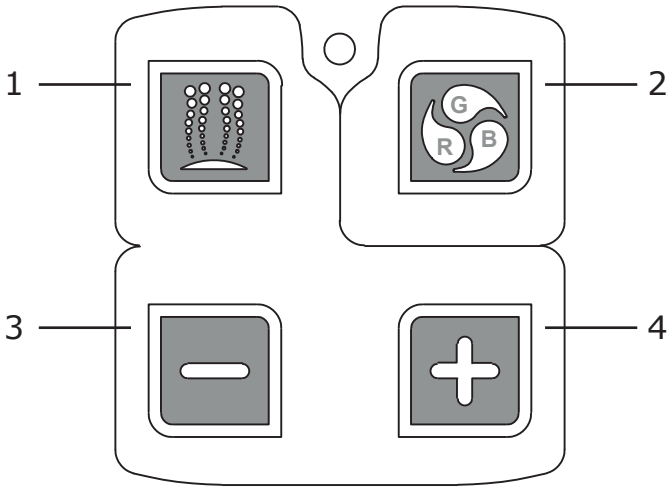
### Horizontal and vertical regulation (footrest)

Horizontal regulation of the position of the footrest by using the two dedicated control knobs placed beneath the footrest.

Vertical regulation of the position of the footrest by using the lever that unlocks the movement of the support.

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#### Blower

*The air inlet system consists in a series of injectors placed on the bottom of the basin.*

#### Operation:

- Turn on the switch that powers the chair.
- Fill half container with hot water not over 38°.
- Start the hydro-massage by pressing the "1" button.
- Intensity regulation by using the "+" and "-" buttons.
- Perform the treatment.
- Turn off the hydro-massage by pressing the "1" button.

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*The Cromorelax mode "Push Up" is controlled by switching on and off an output spot in the following way:*

- *Start the chromotherapy by pressing button number "2" to activate the rainbow cycle consisting of the following sequence of colors: BLUE, YELLOW, GREEN, RED, WHITE, ORANGE, CYAN, VIOLET, WHITE.*
- *If you briefly press the "2" button, the rainbow loop is stopped and, at each additional pressure, a fixed color is selected.*
- *During the rainbow cycle or if a fixed color is selected, the long press for at least 3 seconds of the "2" button will allow the cromorelax to be switched off.*

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### MATERIALS

*Floor: Wood birch plywood stained  
wenge*

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*The multifunctional podiatric chair Saint Barth ensures an excellent comfort to both the customer and the operator, while its simple management in its movements and regulations allow the operator to carry out his work even in narrow places.*

*The chair's functions are adaptable to anybody, and its hypoallergenic padding is entirely washable.*

*The hydro-massage and Cromorelax functions allow a great technical performance and an amazing pleasure for anyone who will experience our new wellness product.*

*The cleaning of the basin is fast thanks to the shower hose, placed at the base of the chair, which allows a quick sanitation.*

#### **BEFORE EACH TREATMENT**

*Fill in the basin minding the water level.*

*Check that the water temperature does not exceed 38°C.*

#### **AFTER EACH TREATMENT**

*Basin cleaning: use common liquid neutral detergent such as soap and a damp cloth  
Do not use ammonia or acetone based cleaning products since their use would permanently damage the chair.*

*Also, do not use formic acid or formaldehyde based products for the sanitation of the basin.*

*For any inquiry please contact your vendor.*

*VISMARA declines any responsibility for any damage caused by negligence regarding what mentioned above.*

*In case of light scratch marks it is possible to sand down the affected areas with type 1000 damp sandpaper and then apply some polish with a soft cloth.*

*Seat cleaning: In order to clean and sanitise the chair after each treatment use a common liquid neutral detergent such as soap and a damp cloth*

*Do not use ammonia or acetone based cleaning products since their use would permanently damage the chair.*

#### **PERIODIC MAINTENANCE**

*It is advisable, for the best chair functioning to have it maintained periodically by your vendor.*

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*The installation has to be carried out by qualified staff, able to certify their work.  
Do not proceed with any treatment to patients with inflamed legs and feet, with open wound, infections or continuous pains.  
In case of even light sickness of the patient stop the treatment immediately.*

*Read the present manual carefully before operating the chair. The use of the podiatric chair is for qualified personnel only.  
The podiatric chair has to be connected to an electric system in compliance with the latest safety standards and provided with grounding device and circuit breaker.  
In case these requirements are not met the use of the chair is forbidden.  
The use of any extension cable or socket has to comply with the technical specs of the product (refer to paragraph "Technical data").*

*Before any treatment please ensure that every part of the chair is intact and that it does not present any sign of wear: if that is not the case stop the treatment immediately.*

*Do not operate with wet or humid hands.*

*Do not touch, operate or start any electric equipment around the basin, the non compliance to this indication can lead to great danger of electrocution.*

*Do not use solvents or chemical additives for the cleaning of the chair.*

*Do not handle flammable liquids nearby the chair.*

*In case of water leaks, unplug the chair immediately.  
Before any cleaning or maintenance operation unplug the chair.  
Unplug the chair when not in use.*

*Keep children away from the chair.*

*Do not touch or tamper with the electric and hydraulic components.*

*Each part of the chair has to be correctly installed.*

*Do not trash the packaging.*

*The company declines any responsibility for fire or damages to people and things.*

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*In case of malfunction please contact your vendor only.*

## Problemi e rimedi / Problems and remedies

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*In case of malfunction, do not touch or tamper with any electric or hydraulic component.*

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*The product is covered by a full warranty of 12 months in presence of the receipt or any other document proving the purchasing date.*

*The right of call and the work hours, to be settled directly with the costumer support, are at expenses of the inquirer, for any intervention requested without actual deficiency of the product.*

*The present warranty does not cover the parts of the chair that are subject to wearing, such as warning lights, protection fuses, wooden parts and any other components that will turn out defective as a result of negligence, carelessness in the use, wrong installation or maintenance, intervention by unauthorized personnel, careless shipping.*

*The warranty is subject to forfeiture also for any circumstance that is not ascribable to production deficiency, such as: lighting, voltage drop, electrolysis, corrosion and in general any cause attributable to the action of limestone residues in the water or any deficiency in the electric and hydraulic system.*

*The manufacturer declines any responsibility for any direct or indirect damage sustained by people, animals and things.*

*The present warranty will not be lenghtened in case of proven malfunction.*

**N° matricola / Serial number**